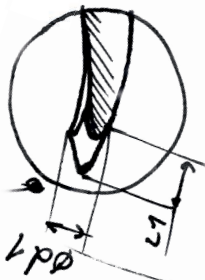
UNICONE

**UNICONE**  
time for change

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## Advantages

### Effectiveness

- + Unique design of the instrument brings quick and safe preparation of a root canal with reciprocal movement
- + Allows for easy obturation with 6% great taper gutta percha points

### Flexibility

- + Made from high flexible Nickel-Titanium with extended lifetime thanks to a special technology treatment
- + Variable triangular cross-section design guarantees ideal combination flexibility and resistance

### Originality

- + Comfortable force and easy to manage by the dentist
- + Gold plated shank for better protection of dental handpieces

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## Description

Its unique construction and material processing predestines the Unicone instrument to highly professional results when treating root canals. Reciprocal movement intensifies the instrument's unique properties.

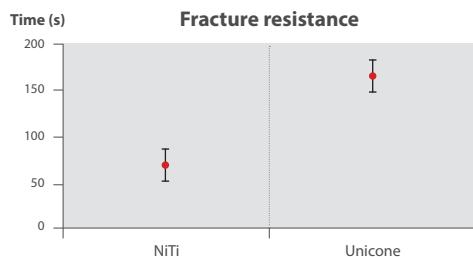
The key instrument is Unicone 6/025. The 6/020 is intended for narrow root canals, and the 6/040 for wide root canals.

Unicone has an inactive tip designed based on our experience gathered while developing our previous successful systems (WIZARD CD Plus and WIZARD Navigator), the risk of root canal perforation is thus further eliminated.

Special heat-treated NiTi material increases Unicone's flexibility and durability.


## New refined NiTi material

The newly developed procedure improves both the instrument's flexibility and brittle fracture resistance.

Comparison of a instrument made of a standard NiTi alloy with the one made of the refined alloy by MEDIN (artificial canal – 5 mm radius; 60 ° angle)

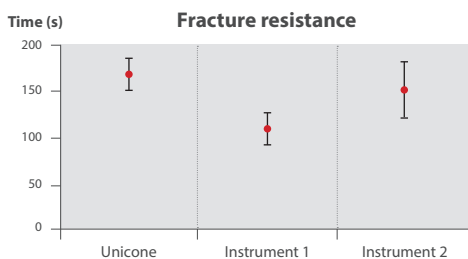Comparison of the Unicone with other reciprocal instruments available on the market; in an artificial canal (5 mm radius; 60 ° angle)

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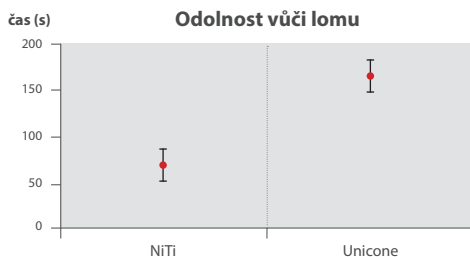
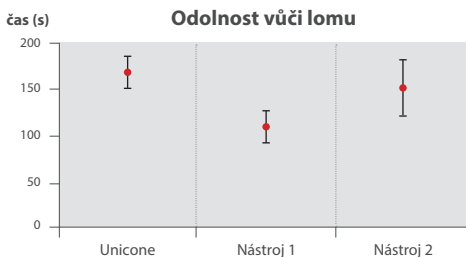
## Step by Step

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1. Primary clearing of a root canal using W-File 010.
2. Measuring of working length – Apexlocator, X-ray.
3. If required – widening using hand instruments K-File 015 to 020.
4. Preparation using Unicone 6/025 without or with a maximum of 2 to 3 in-and-out pecking movements. The endo motor may be set to any of the reciprocal movement programmes available.
5. After the preparation of approx. 1.5–2 mm, clean the instrument and rinse the canal with a disinfectant solution. For anatomically complicated canals, check the passage using the W-File instrument.
6. Repeat steps 4 and 5 until the working length is reached.

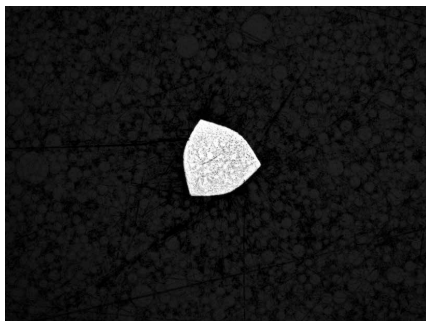
Wide canals may be enlarged using Unicone 6/040. Once the instrument stops to penetrate, it is crucial to clean both the instrument and the canal immediately, possibly enlarge the canal using the Unicone 6/020. For complicated root canals primary clearing both WIZARD Navigator W1 and WIZARD Navigator W2 by MEDIN can be used. Similarly, for narrow canals, it is possible to use Unicone 6/020 and fill the root canal with the ISO 6/020 gutta percha point. For wide canals (incisors, canine teeth), steps 1 and 3 may be leave out. After using the instrument, place it in a cleaning box and rinse the root canal with the solution carefully.

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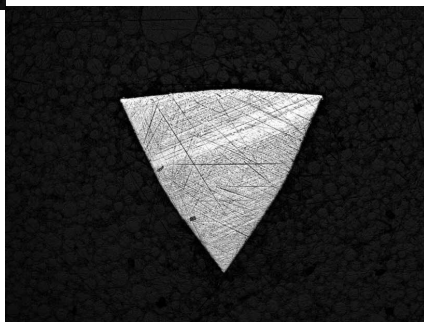
Instrument profile 9 mm from the tip /  
Profil nástroje 9 mm od špičky

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## Dentists feedback

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"I have found another good quality Czech-made instrument for endodontics. I appreciate its reusability and compatibility with all currently available endo motors providing reciprocal counterclockwise movement. Variable angle of the cutting edge reduces the tendency to screw the instrument into the root canal. The life of the instrument increases treatment safety and reduces costs. Unicone system performs better in certain aspects than its competing products."

**Pavel Hyšpler, MD – Head Physician**

The Military University Hospital Prague

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**MUDr. Pavel Hyšpler – primář**

ÚVN Praha

"Working with Unicone instruments is safe and comfortable, proceeding with the instrument is easy to control. Unicone instruments allow me to quickly achieve sufficient extension of the prepared root canals for efficient lavage and thereby significantly reduce the time of treatment."

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