CZ

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READ CAREFULLY!

REUSABLE SURGICAL INSTRUMENTS

- Warning**
- Stainless steel surgical instruments cannot be inserted into the physiological saline (NaCl solution), because a longer contact causes corrosion such as a pitting corrosion and stress corrosion.
 - Cleaning solutions in which hydrogen peroxide is added and/or solutions for cleaning with higher alkalinity may lead to a change in colour. This could cause damage to the identification of the surgical instruments.
 - It is necessary to avoid a usage of the alkaline solutions and the hydrogen peroxide solution and the use of oxygenated water when washing the surgical instruments with the carbide part (e.g. needle holder).
 - Do not use any abrasive cleaning materials such as steel wool or abrasive detergents to clean the surgical instruments. If possible, always clean the surgical instruments and sieves in the washing and disinfecting facility for surgical instruments according to the standard ISO 15883-1.
 - The long, narrow cannulas, blind holes and complex parts require a special attention when cleaning.
 - Sterilising parameters apply only to surgical instruments which are sufficiently cleaned.
 - The sterilization parameters apply only to well-assembled, maintained, calibrated and complying equipment for reprocessing in accordance with ISO 15883-1 and 17665 standards.
 - The surgical instruments are supplied NON-STERILE.
 - Remove the surgical instrument from its packaging before first use and remove covers from the sharp ends. These covers are not intended to be sterilized. Place the covers into the waste sorted in accordance with the Catalogue of waste – 18 01 07 – chemicals that do not contain hazardous substances. It is necessary to return the instrument to the manufacturer for repackaging and control in case of impaired packaging. Dispose the package according to the recycling symbols.
 - Perform cleaning, disinfection and sterilization according to the regulation before first use (points 16.–24. of these instructions).
 - Clean and sterilize the surgical instruments as soon as possible after their usage.

13. Restrictions on re-use

- Lifetime of the surgical instruments is determined by the wear and tear and damage due to their usage.
- Repeated cycles of cleaning, disinfecting and sterilization have minimal impact to the surgical instruments.

INSTRUCTIONS

14. Place of use

- Wipe the blood and any remaining tissues from the surgical instruments during the surgery before they are allowed to dry to the surface.
- Rinse the surgical instruments with openings by the sterile or purified water before the impurities and/or remaining tissues are allowed to dry inside.
- Soiled surgical instruments have to be separated from the clean ones to avoid contamination of unused surgical instruments, staff and surroundings. Contaminated surgical instruments must be separated from medical waste.
- The surgical instruments should be covered with the drape dampened in the sterile or purified water to avoid drying of blood or debris.

15. Storage and transport

- Soiled surgical instruments have to be transported separately from the clean ones to avoid their contamination.

16. Preparation for cleaning

- Open the surgical instruments with toothed or threaded latches and hinges. Separate the pointed and sharp surgical instruments. Wipe the oil from the locks using a cloth at the new surgical instruments. A larger amount of oil in the locks of the new surgical instruments could lead to the formation of spots on the surgical instrument surface after sterilization.
- Remove the sharp removable parts of the surgical instruments prior the manual cleaning or place them onto the separated tray.
- Disassemble or loosen all connections before processing of the surgical instruments. You will get further detailed information about the dismantling of the surgical instruments from your sales representative of MEDIN.

17. Cleaning and disinfection – manual

- Place MEDIN surgical instruments into the washing agent carefully to avoid moving of them freely during washing process or their overlapping which could damage their surface.
- Remove the surgical instruments from the disinfecting agent and rinse under water after the expiration of recommended time
- Wash the surgical instruments manually, remove coarse dirt mechanically with brush in fresh cleaning and disinfection solution in an ultrasonic bath, and rinse under tap water.
- MEDIN company does not prescribe the use of a specific detergent and disinfectant.
- The washing agent and disinfectant have to be used in accordance with the instructions of the manufacturer. Keep prescribed concentration, temperature, water quality and action time when using.
- Remove the washing agent under tap water after the expiration of recommended action time.

18. Cleaning and disinfection – automatized

- The validated machine cleaning and disinfection procedure must always be preferred to manual cleaning because of the higher safety of the process. A good cleaning is also a requirement for successful sterilization.
- We recommend washing the surgical instruments before sterilization using automatic washers with thermal disinfection.
- Place MEDIN surgical instruments into the disinfection agent carefully to avoid moving of them freely during washing process or their overlapping which could damage their surface.
- MEDIN company does not prescribe the use of a specific detergent and disinfectant.
- The washing agent and disinfectant have to be used in accordance with the instructions of the manufacturer. Keep prescribed concentration, temperature, water quality and action time when using.

19. Thermal disinfection

- Disinfect at a minimum temperature of 90 °C for at least 5 minutes in case of cleaning in washer.

20. Drying

- If drying cycle is not included in the manual washer: dry each surgical instrument properly.

21. Check

- MEDIN surgical instruments have to be checked after treatment and before sterilizing for the following:
 - Cleanness.
 - Any damage including corrosion, change of the colour, excessive and significant scratching, chipping, wear and cracks
 - Proper functioning including the sharpness of the cutting surgical instruments, flexion of the elastic parts of the surgical instruments, moving of the joints / joint connections / threaded latches and other moving parts, such as handles, toothed latches and connections.
 - Missing or removed (scraped) numbers of the parts and wear and tear.
 - Discard the surgical instruments with the aforementioned defects.
- Make sure that the surgical instruments have an intact surface and that they are properly adjusted and functional. Do not use the surgical instruments that are severely damaged, have illegible marks, show the signs of corrosion or have dull blades. Discard these surgical instruments. For more detailed instructions on the check of functionality, please contact your MEDIN sales representative.

22. Maintenance

- Maintenance and care is usually performed before the test function. Maintenance and care means a targeted application of the preservatives onto the surgical instruments into the joints, hinges or threads and sliding surfaces, e.g. at the staples, scissors, punches after a thorough cleaning and disinfection. This prevents friction of a metal on a metal and consequent friction corrosion. The surgical instruments are being maintained in a movable state. Requirements for the preservatives for the surgical instruments: paraffin- / white oil-based, complying with the applicable European Pharmacopoeia or the United States Pharmacopeia, biocompatible, suitable for the steam sterilization and vapour-permeable The surgical instruments must not be treated with silicone preservatives. They may complicate movement and reduce the effect of the steam sterilization.
- A dismantled surgical device should be assembled prior to sterilization unless otherwise indicated.
- The manufacturer only is entitled to perform service interventions.

23. Packaging

- Put the disinfected, cleaned, rinsed and dry instruments in their place in a sieve. In addition, use a suitable sterilization packaging or a reusable hard container such as a system of sterile barriers according to ISO 11607 standard.
- It is necessary to pay attention to the protection of the surgical instruments and ensure that the spiky or sharp surgical instruments do not come into contact with other objects whose surface could be damaged.
- Pack individual surgical instruments, for which MEDIN storage is not available into the sterilization bags that meet the requirements of ISO 11607.

24. Sterilization

- Sterilize cleaned and disinfected surgical instruments only.
- It is recommended to use sterilisation with wet heat in a steam sterilizer at a temperature of 134 °C and the time of sterilization of 7 min. and pressure of 310 kPa for the surgical instruments.
- When using of hot air sterilization is recommended in a special precaution, it is performed in the facilities with forced air circulation at the temperature of 160 °C for 60 minutes or at the temperature of 170 °C for 30 minutes or at the temperature of 180 °C for 20 minutes.

25. Storage

- Store the packed surgical instruments in a dry clean environment without extreme temperature and humidity, away from direct sunlight, animals, insects, dust, fungi and chemicals.

26. Other information

- The process of washing with thermal disinfection has been verified by MEDIN company. In case of use of some other methods of the washing procedure than recommended, MEDIN company does not guarantee the result. When using some other method of washing than recommended, MEDIN company recommends this process to be validated by the supplier of such washing facility.
- The MEDIN company has validated the method of sterilization with the damp heat at the temperature of 134 °C and the time of the sterilization of 7 min and pressure of 310 kPa for the representatives of the WORST-CASE MEDIN products.
- The MEDIN company has validated method of dry heat sterilisation for the representatives of the WORST-CASE MEDIN products.
- The instructions for cleaning and sterilization are stated in accordance with the norms and standards ISO 15883, ISO 17664, ISO 17665-1. The instructions listed above have been confirmed by the manufacturer of the surgical instrument as appropriate for the preparation of MEDIN non-sterile surgical instruments. It is the responsibility of the provider to achieve the desired outcome by preparing a product with the use of the equipment, materials and workers on the pre-site. This requires the validation and routine monitoring of the procedure. Likewise every deviation of the provider from these recommendations should be properly assessed in terms of the efficiency and any possible undesired consequences.

27. Disposal of the surgical instrument

- A discarded surgical instrument is considered as a hazardous waste. The user is responsible for the implementation of the measures to the safe handling and disposal of the product.
- Damaged surgical instruments are disposed of after decontamination, washing and drying as a potentially hazardous waste according to the Catalogue of waste – Group 18 – medical waste.

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