**CZ**

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**EN**

Information on use PL0012  
READ CAREFULLY!

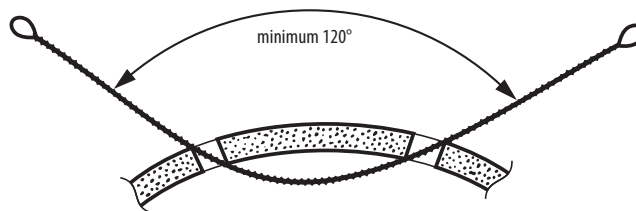
**WIRE SAW OLIVECRONE****PURPOSE**

Mostly used during neurosurgery to create an opening in a part of the skull.

**INSTRUCTIONS FOR USE**

**127 17 0011** (length 32 cm), **127 17 0021** (length 40 cm)

Please keep minimum angle of 120 degrees between tight ends of the wire saw Olivecrone during the use.

The cutting effect of the saw is determined through the spiral winding of two core wires, which are equipped with the winding of the cutting wire all over the length. The wire edges execute the function of the teeth of the foliate saw by cutting. There are strengthened eyelets on the end of the saws, which are inserted in the eyelet of the holders. They are mostly used in neuro-surgery. The saw is extended through two pre-drilled holes in the cranial bone, both ends will be fixed in the holders. The bone between the holes is incised by movement and traction.

For insertion of saws use: Insertor of wire saws **133 17 0061** (33 cm), **133 17 0070** (25 cm)

To grasp saws use: Holder of wire saws fix (couple) **127 17 0051**

**CAUTION****Limit of repeated treatment**

Checking of the strength of the wires before each use.

**INSTRUCTIONS**

**Place of use:** Specialized facility for health-care providers.

The procedures of cleaning, disinfection, drying and sterilization can be found in general instructions PL0169.

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