CZ

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**EN**

Information on use PL0037
READ CAREFULLY!

EYE PROBES, EYE EXPLORERS, LACHRYMAL DILATATORS

PURPOSE

They are used in ophthalmology – by examinations, by operations and dilatations of eye ducts. They are especially fine instruments and they require fine manipulation. The sizes of functional parts are marked on the instrument in below mentioned way.

INSTRUCTIONS FOR USE

Dilatator of lachrymal points

Small instrument handle, knurled with one to three plunging cuts on the handle that converts through the cone into the functional part. The total length of the instrument is 75 mm.

dimensions in the spike of rounded functional part (diameter):

- ∅ 0.10 mm number of plunging cuts on the handle: 1
- ∅ 0.25 mm number of plunging cuts on the handle: 2
- ∅ 0.40 mm number of plunging cuts on the handle: 3

The size is marked with the number of plunging cuts on the handle.

Eye explorer Bowman size 1–3

The instrument is a rod of various diameter, in the middle with flat handle. The total length of the instrument is 130 mm.

dimensions of functional part (diameter):

- ∅ 0.5 mm size 1
- ∅ 0.7 mm size 2
- ∅ 0.9 mm size 3
- ∅ 0.5 mm (on the first side), ∅ 0.7 mm (on the second side) size 1; 2

The size is marked with size number on the handle.

Infantile eye explorer

The instrument is double-sided, the cylindrical part (∅ 2 mm) is used as a handle, it converts with the cone into fine rounded ends. The total length of the instrument is 120 mm.

dimensions of functional part (diameter):

- ∅ 0.5 mm

The size is marked according to ∅ of the end on the handle.

Eye probe

The instrument is double-sided, the cylindrical part (∅ 3.3 mm) is used as a handle, it converts with the cone into the length of 17 mm into fine rounded ends. The total length of the instrument is 138 mm.

dimensions of functional part (diameter):

- ∅ 0.3 mm (on the first side), ∅ 0.8 mm (on the second side)

The size is marked according to ∅ of the end on the handle.

INSTRUCTIONS

Place of use: Specialized facility for health-care providers.

The procedures of cleaning, disinfection, drying and sterilization can be found in general instructions PL0169.

Manufacturer: CZ43378030 Number and date of last revision: R01/2011-12-22
MEDIN, a.s. | Vlachovická 619 | CZ 592 31 Nové Město na Moravě | Czech Republic
tel.: +420 566 684 336 | fax: +420 566 684 385 | e-mail: export@medin.cz

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