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ÚČEL POUŽITÍ

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CURETTES, UTERINE CURETTE

PURPOSE

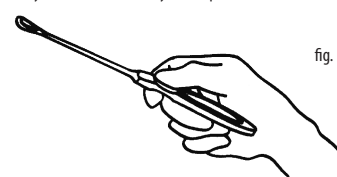
Curettes are used to scrape out tissue, mucous membranes, or to obtain a histological preparation.

INSTRUCTIONS FOR USE

Prior to each reusing of the instrument, it is necessary to check the quality of the cutting edge, best with a magnifier and on a nail (it must not be blunt, turned or corrugated, must not slip on the nail). Curettes are held by a handle, and by introducing slight pressure the working part scrapes off the tissue. With curettes for biopsy (non-sucking), we scrape off a narrow strip of the mucous membrane which is then sucked from the cavity of the curette by means of a bigger syringe.

When performing uterine curettage with sharp curettes, it is necessary to follow an extremely careful procedure:

The curette should be always inserted in the direction of the uterine axis. If the uterus is in anteversion, the cutting edge is directed to the front; if the uterus is in retroflexion, it faces the rear end. The curette should be held lightly with three fingers, according to Figure 1, and inserted carefully until it touches the uterine fundus. It should never be done quickly or with too much force as otherwise, the uterine wall could be broken (this is especially easy e.g. in cancer of the uterine body). Then we place the cutting edge of the curette against the uterine wall and by introducing slight pressure we draw the curette up to the inner cervical isthmus (opening).

Using this technique, we gradually scrape off the whole of the front wall of the uterine cavity. Then we turn the curette and do the same on the back wall. Using several diagonal moves, we scrape the uterine fundus and corners as well. The scraped off uterine mucous membrane exits through the cervical channel into the cavity of the vaginal speculum, where it can be wiped off. As soon as we can hear a grating sound during the subsequent move with the curette against the uterine wall or when we feel that the uterine wall puts up more resistance to the curette, we must stop the curettage immediately. If a crumbly mass is detected during the curettage, even macroscopically suspected to be cancerous tissue, we should stop the operation immediately so as not to cause perforation. This may occur under the following circumstances:

- 1) The operator is not well informed about the uterus' position and, therefore does not insert the curette in its longitudinal axis and meets resistance at the site of the cervical isthmus, uses more force and breaks through the uterine wall.
- 2) The operator is not well informed about the uterus' size and presumes that the uterus is bigger than it is in reality. By more invasive insertion of the curette he may break through the uterine wall.
- 3) The operator has used a curette of a size unsuitable for that of the uterus and cervix. If he uses a wider curette, he meets resistance at the inner isthmus of the cervix and in an attempt to overcome it he may widen the isthmus. If the curette is too small, he may cause perforation as it is easier to penetrate the muscle wall in this case as the tip of the functional part of the curette puts up less resistance when in contact with the uterine fundus.
- 4) The uterine wall is thin or too soft due to a pathological process.

In uterine curettage with blunt curettes (obstetric operations, interruption of pregnancy), the method of holding and the technique of curettage are similar. The uterine wall is free of mucous membrane and chorion as soon as the operator hears the grating sound caused by scraping the curette against the uterine muscles. Time and again during curettage, it is necessary to clear away the scraped off contents out of the uterus through the cervix.

CAUTION

Limit of repeated treatment

It is necessary before repeated use to check the quality of the blade, preferably using a magnifying glass and on the nail (it must not be dull, bent or wavy, and it must not slip along the nail).

INSTRUCTIONS

Place of use: Specialized facility for health-care providers.

The procedures of cleaning, disinfection, drying and sterilization can be found in general instructions PLO169.

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