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**DEPTH GAUGES AND GAUGES****PURPOSE**

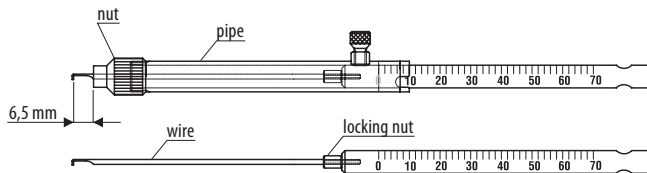
Depth gauges and gauges are auxiliary tools used to determine the parameters of various body organs, instruments, implants or bone grafts.

**INSTRUCTIONS FOR USE****1. Depth gauge 129 79 8900 and 129 79 8910**

The instrument is used during operations, bone remedies and helps to choose the correct size of the screw according to a drilled opening in a bone. The wire of the depth gauge is led to the pre-drilled opening in the bone, leans on the bottom or gets hooked outside the drilled bone; the required size of the screw is read on the scale (including the head). When using plates or washers, it is necessary to choose the screw longer by the thickness of the plate or of the washer, deducting the countersinking of the screw's head.

**2. Change of the spare part 934 40 0742 in the dept gauge 129 79 8900**

- unscrew the nut and take the instrument out of the pipe
- loosen the locking nut on the wire and unscrew the wire out of the dept gauge and then twist the nut off the wire
- twist the locking nut on a new wire, screw the wire back and lock with the nut
- put the instrument back on the pipe and lock with the nut
- check if at nonius setting 10 mm, the size of the hook is 6.5 mm above the nut
- in case the size does not agree, adjust the screwing-in of the wire in the depth gauge

**3. Gauges 129 79 9050 and 129 79 9060**

The instrument is used during operations, bone remedies and helps to choose the correct size of the cannulated screw according to a drilled opening in a bone.

The instruments put closely to a bone and a wire, measure the depth of screwed-in wire. It is essential to use wire 160 mm (129 79 9030) for correct length deducting for the instrument 129 79 9050. It is essential to use wire 300 mm (129 79 9040) for correct length deducting for the instrument 129 79 9060.

**INSTRUCTIONS**

**Place of use:** Specialized facility for health-care providers.

The procedures of cleaning, disinfection, drying and sterilization can be found in general instructions PL0169.

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