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#### Information on use PLO209

#### READ CAREFULLY!

### ROOT INSTRUMENTS WIZARD Navigator

#### Instruction for use:

- Probing and the primary penetration into the root canal is carried out with the handheld ISO 006-010 size W-File according to the estimated width of the root canal to be treated. After penetrating the root canal we set about expanding it.
- Prepare the root canal using the W-1 instrument (purple stripe) and vertical filing movements, applying a slight apical pressure, up to 1/2 or 3/4 of the length depending on the root canal's configuration. The entire preparation is carried out using a lubricant and should not last longer than 10 s. The W-1 instrument is placed in the cleaning box and the root canal rinsed thoroughly with the solution.
- As with the W-1 instrument, use the W-2 instrument (white stripe) to prepare the same length with a slight pressure. Place the W-2 instrument into the cleaning box and rinse thoroughly.
- Penetrate a safe length into the root canal using the W-1 instrument (purple stripe). Rinse and penetrate the same length with the W-2 instrument (white stripe).
- Repeat the above procedure until a safe length of the root canal is attained. The preparation is completed using the W-2 instrument. If necessary, the entrance to the root canal can be widened or moved using the W-XN instrument.
- Measure the working length of the root canal (X-Ray, Apexokátor). Rinse thoroughly and using a lubricant and the W-2 instrument prepare the set working length.
- Place the W-3 instrument (yellow stripe) on the end and, using a lubricant, prepare the entire length of the root canal. Place the instrument in the cleaning box and rinse thoroughly.
- If the root canal's configuration so requires, it can be filled using your preferred method.
- For the majority of root canals the W-4 instrument (red stripe) is used to further expand the apical width to 025, or the W-25 (blue strip) is used to expand it to a width of 030.
- The canal prepared in this manner is now filled.
- If more massive working is required the W-5 instrument (blue stripe) can be used to continue to an apical width of 030, then the W-6 instrument (green stripe) can be used to a width of 035 with a 6 % taper or W-7 (black stripe) to a width of 040 with a 6 % taper. For less weakening of the root canal walls it is also possible to use the W-8 instrument (green stripe) to a width of 035 and the W-9 (black stripe) to a width of 040 with a 4 % taper.
- We strongly recommend only using Medin brand accessories for the Wizard Navigator endo system.

#### CAUTION

- Don't use agents containing phenol and hydrogen peroxide  $H_2O_2$  for disinfection.
- Instruments made of NiTi can be disinfected with 5% solution of sodium hypochlorite NaClO, however not longer than 5 minutes.
- Attention, the endodontic instruments are sharp!
- Avoid the use of alkalic (PH higher than 8), acidic (PH lower than 6), sodium hydroxide solutions or solutions containing mercury salts for instruments and boxes made of aluminium and aluminium alloys.
- The cleaning away of clinging rests of tissues is done with compressed air, sluiced with fully desalted or distilled water.
- Use instruments for root canals treatment can cause a package perforation, this is the reason why they must be put in safety solid packages – PET bottles, canisters, etc. during the waste classification.
- Only dentists can work with the instruments.
- The producer is not responsible for health or possessions injuries.
- The instruments contain nickel that can cause allergic reaction of the patient to this metal.
- Keep recommended torsion settings.

#### Limit of repeated treatment

The end of lifetime is determined through their wear and damage by using.

#### Instructions for disinfection, cleaning and sterilization of dental rotary instruments

**Sphere of using:** From hygienic and safety reasons must all instruments be disinfected, washed and made sterile before intra-oral use.

**Disinfection:** Put the instruments into disinfection agent immediately after their using or as soon as possible. Disinfection agent must be used in conformity with instructions mentioned on the producer's label. For the preparation of disinfection and washing solutions respect the procedure given by the producer. High concentration can result in appearance of chinks or lead to damage of coloured distinguishing rings. Respect the disinfection given time.

*Recommended agents:* Sekusept Pulver, Neodisher Septo MED.

**Preparation for cleaning:** No specific requirements.

**Cleaning by hand:** Clean away the rests of dentine with a soft brush, sluice and dry the instruments. After drying, remove possible clinging dirt with compressed air.

**Check:** Carefully check and put away all defected, bent, broken, blunt, corroding or in other way damaged instruments. Put the checked instruments into dishes or stands. Treat the instruments made of carbonaceous steels with an anticorrosive agent.

**Automated cleaning:** Carefully respect the instructions and data of advised cleaning agent concentrations. Read attentively all warnings in the introduction of instruction for use. Carefully check the instruments after finished washing cycle likewise after hand washing.

**Drying:** If the drying is a part of washing and disinfection device cycle, the temperature of 120°C mustn't be exceeded. Protect the instruments made of carbonaceous steel with an anti-corrosion agent.

**Packing:** The instruments will be put separately into packages or into sterilizing dishes that are wrapped after it. Sterilize separately the stands with instruments, open the boxes and sterilize them also individually. The standard packing material can be used. It is ensured that the package is big enough and the instrument can be put in it without seal stress. Write over the packages.

**Sterilization:** If it isn't mentioned in „Special instruction“ otherwise, we recommend the sterilisation with wet heat in steam sterilizer equipped with antibacterial filter at the temperature of 121 °C, overpressure of 205 kPa and for 20 minutes or at the temperature of 134 °C, overpressure of 304 kPa for 10 minutes. If it is mentioned in „Special instructions“ for hot-air sterilization it is performed in devices with forced air circulation at the temperature of 160 °C for 60 minutes or at the temperature of 170 °C for 30 minutes or at the temperature of 180 °C for 20 minutes, if it isn't mentioned otherwise. The hot-air sterilizer is opened later on after cooling down to 80°C when the cycle is finished. Packed sterilized instruments can be sterilized repeatedly.

**Storage:** Store in dry places and hinder the repeated contamination.

**Other information:** The joint-stock company MEDIN validated the above mentioned methods of cleaning, disinfection, drying and sterilization. Through the validation were all procedures recognized as qualified for the preparation of medical instruments for repeated using. The user is responsible for ensuring of necessary materials, equipment and staff training in accordance with these specified procedures to reach the required effect. The user can use some other methods, but the efficiency of these alternative methods must be verified by specialized laboratory.

**Recommendation:** Wizard Navigator instruments with working part taper ratios of 4%, 5%, 6% and 7% are intended for the machine-assisted preparation of root canals. The nickel-titanium alloy used to make the instruments guarantees their high flexibility. The instrument's tip is suitably rounded to reduce the risk of perforating the root canal.

Recommended setting for endomotor			
Wizard Navigator	Marking	Revolutions (R. P. M.)	Torsion (N.cm)
W-1	4/010	140–180 min <sup>-1</sup>	1,2
W-2	5/015	140–180 min <sup>-1</sup>	1,2
W-3	6/020	140–180 min <sup>-1</sup>	2,1
W-4	6/025	140–180 min <sup>-1</sup>	2,1
W-XN	7/025	140–180 min <sup>-1</sup>	2,3
W-5	6/030	140–180 min <sup>-1</sup>	2,3
W-8	4/035	140–180 min <sup>-1</sup>	2,3
W-6	6/035	140–180 min <sup>-1</sup>	2,3
W-9	4/040	140–180 min <sup>-1</sup>	2,3
W-7	6/040	140–180 min <sup>-1</sup>	2,3

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