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**INSTRUMENTATION FOR HUMERAL NAILS****Description of the instrumentation**

Instrumentation is used to lock the nail in the humeral bone.

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**Compatibility**

Instrumentation is used with humeral nails, plugs and flat head screws.

PLO224 – short humeral nail, long humeral nail

PLO225 – Flat head screws

PLO226 – Plugs

Instrumentation is stored in the basket:

Instrumentation for humeral nails – 397 139 09 0630

**Indication**

Instrumentation is intended for the insertion and locking of humeral nails.

**Contraindication**

Contraindications for the use of instrumentation are consistent with contraindications for humeral nails, see PLO224

**Instructions for use****Preparation of the aiming device**

Attach aiming arms to the base body of the aiming device in a position depending on the operated limb. Aiming arms are fitted with fixing screws, which are adequately tightened to the base body by tightening rod.

**Installation of aiming device and nail insertion***Assembly of the aiming device with the nail*

– The nail is attached to the grooves of the aiming device and securely fixed with a screw of the aiming device.

*Assembly of the aiming device with sleeves and checking the aiming device with nail*

– The position of aiming arms to locked nail is checked before use. The check is performed for all seven holes in short nails, and all twelve holes in long nails.

**Preparation of instruments**

Before the surgery it is necessary to check the completeness and functionality of all instruments included in the instrumentation.

**WARNING**

Crossbars of the aiming device are made of X-ray translucent material. Sterilization can only be performed by steam at a temperature of 134 °C! Using higher temperatures may result in permanent deformation and loss of functionality of crossbars!

Products may not be combined with products by other manufacturers.

**Reprocessing limitation**

Repeated processing has minimal effect on these instruments. End of life is normally determined by their wear and damage during use.

**INSTRUCTIONS****WARNING:**

With steel instruments, it is necessary to avoid agents containing chloride ions, which can cause corrosion.

For instruments with components made of aluminum and aluminum alloys it is necessary to avoid alkali (sodium hydroxide and sodium carbonate) and acidic solutions of pH below 6.

In case of instruments with plastic parts, it is possible to use the same agents as for all-metal instruments, if not stated otherwise in a special warning.

**Place of use:**

Professional workplaces of health care providers, namely surgical, trauma and orthopedic departments.

**Storage and transport:**

Prevent deformation and scratches to surgical instruments; avoid their contact with other metallic materials and chemicals.

**Preparation for cleaning**

When cleaning and disinfecting, it is necessary to avoid agents containing chloride ions, which can cause corrosion of steel parts.

1. Removable the arms from the body of aiming device.
  2. Mechanically clean with water using a brush.
  3. Rinse instruments with pressurized water.
  4. Leave the instruments in a disinfectant solution for 20–30 minutes. Recommended disinfectants are Sekusept Pulver, NeodisherseptMED. In case of using another solution you risk damaging the instruments.
  5. Rinse repeatedly with pressure water jet until there is no unpolluted water. It may be performed in conventional pressure washers with instruments connected to the nozzles.
- Blow the instruments with compressed air

**Disinfection:**

Either device for washing and thermal or thermochemical disinfection is recommended, or hand washing of implants after chemical disinfection with virucidal effect, while the two stages can be combined using disinfectants with combined washing effect. Disinfectant can be used in accordance with the instructions on the label of the product. In the preparation of disinfectants and cleaning solutions, the procedures specified by the manufacturer must be followed. After disinfection, the implants are rinsed with demineralized water to remove chemical residues remaining on the implant.

*Recommended products:* Sekusept pulver, NeodisherseptMED**Drying:**

If drying is part of cycle for washing and disinfecting device, a temperature of 120 °C should not be exceeded.

**Maintenance:**

After each surgery, individual parts of the instrumentation must be checked. It is also necessary to check whether cutting instruments are blunting. In the event of damage, the respective parts must be supplied to the manufacturer for repair. After 10 to 20 successful surgeries, you are advised to contact the manufacturer for a maintenance check of the instrumentation.

**Inspection and testing of function:**

All parts of instrumentation must be checked before use. Functionality check of connection to the nail must be performed.

**Sterilization:**

Unless otherwise stated, it is advisable to use steam sterilization in a steam sterilizer equipped with antibacterial filter at 121 °C, pressure 205 kPa for 20 minutes or at 134 °C, pressure 304 kPa for 10 minutes.

Crossbars of the aiming device are made of X-ray translucent material. Sterilization can only be performed by steam at a temperature of 134 °C! Using higher temperatures may result in permanent deformation and loss of functionality of crossbars!

**Other information:**

The company MEDIN, a.s. has validated the above procedures for cleaning, disinfecting, drying and sterilization. Within the validation these procedures were recognized as eligible in the preparation of medical devices for reuse. User is responsible for ensuring the necessary materials, equipment and staff training in accordance with these codes of practice, to achieve the desired result. The user can also employ other methods, but the effectiveness of these alternative procedures must be verified by professional laboratory.

**Storage:**

The instrumentation must be stored in a dry place without direct exposure to chemicals.

**Disposal**

Instrumentation is considered a hazardous waste. User is responsible for implementing measures of safe handling and disposal of the product.

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