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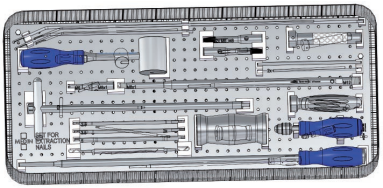
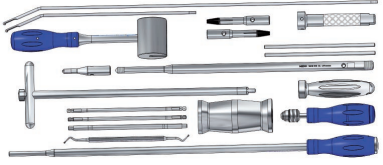
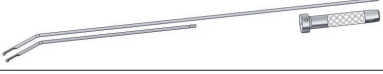
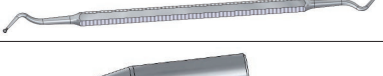
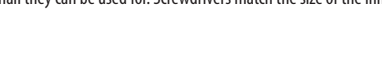
**SET FOR EXTRACTION OF NAILS****Description of the instrumentation**

The set is intended for extraction of damaged nails, which cannot be removed with common procedures.

Instrumentation comprises:		
Pcs	Name	Image
	Set for extraction of nails 397139090660	
	Instrumentation for extraction of nails 397129697855	
1	Handle 397129695131	
1	Extracting rod, handle, weight 397129691610	
1	Extractor of broken cannulated nails 397129991012	
1	Excavator double-sided 397148510100	
1	Connecting piece M6_M8 397129697820	
2	Tightening rod 397129696100	
1	Punching rod 397129697840	
1	Mallet 400 g 397129696060	
2	Extractor M6_M8 397129697830	
1	Key 397129691590	
2	Hex screwdriver, 2.5 mm 397129695231	
2	Hex screwdriver, 3.5 mm 397129695251	
2	Hex screwdriver, 5 mm 397129695273	

Compatibility

Screwdrivers are equipped with AO terminal extension, which can be fixed in a drill or AO terminal handle. All instruments are marked with the size of the nail they can be used for. Screwdrivers match the size of the inner hexagon of the screw.

Indication

The set is intended for the extraction of broken or otherwise damaged nails.

Contraindication

Health condition of the patient, which does not allow invasive procedures.

Instructions for use**DAMAGED NAIL THREAD**

Use excavator to clean the thread. Insert the M6_M8 extractor into the nail and pull to make sure that it fixes the nail. Then put on the pull rod, mallet and handle and to try to extract the nail with light strokes.

BROKEN CANNULATED OR FULL NAIL

If the cannulated nail is broken, do not pull the nail in parts, but try to extract the nail as a whole. Screw on the handle of the extractor to the selected length of the extractor and extract the nail with light strokes using a 400g mallet. If the nail cannot be extracted in whole, or it is a full nail and some parts cannot be extracted with extractor, it is possible to use a punching rod and knock the remainder of the nail on the side opposite to the side of nail insertion.

EXTRACTION OF INTACT NAILS

Use excavator to clean the inner thread if the nail. Screw an extracting rod with mallet and handle into the M8 nails with inner thread and extract the nail with light strokes. If you are going to extract the M6 nails with inner thread, screw the M6_M8 pulling connecting piece on the rod and then again, knock the mallet, handle and nail out.

WARNING

Products may not be combined with products by other manufacturers.

Reprocessing limitation

Repeated processing has minimal effect on these instruments. End of life is normally determined by their wear and damage during use.

INSTRUCTIONS

WARNING:

With steel instruments, it is necessary to avoid agents containing chloride ions, which can cause corrosion. For instruments with components made of aluminum and aluminum alloys it is necessary to avoid alkali (sodium hydroxide and sodium carbonate) and acidic solutions of pH below 6. In case of instruments with plastic parts, it is possible to use the same agents as for all-metal instruments, if not stated otherwise in a special warning.

Place of use:

Professional workplaces of health care providers, namely surgical, trauma and orthopedic departments.

Storage and transport:

Prevent deformation and scratches to surgical instruments; avoid their contact with other metallic materials and chemicals.

Preparation for cleaning

When cleaning and disinfecting, it is necessary to avoid agents containing chloride ions, which can cause corrosion of steel parts.

- Mechanically clean with water using a brush.
- Rinse instruments with pressurized water.
- Leave the instruments in a disinfectant solution for 20-30 minutes. Recommended disinfectants are Sekusept Pulver, NeodisherseptoMED. In case of using another solution you risk damaging the instruments.
- Rinse repeatedly with pressure water jet until there is no unpolluted water. It may be performed in conventional pressure washers with instruments connected to the nozzles.

Blow the instruments with compressed air

Disinfection:

We recommend either the equipment for washing and thermal or thermochemical disinfection or hand washing of instruments after chemical disinfection with virucidal agent, while the two stages can be combined using disinfectants with combined washing effect. The disinfectant may only be used in accordance with instructions on the label of the product. In the preparation of disinfectants and cleaning solutions, the procedures specified by the manufacturer must be followed. Implants that are designed for disassembly are disassembled and each part is further considered as a separate instrument.

Recommended products: Sekusept pulver, NeodisherseptoMED

Drying:

If drying is part of cycle for washing and disinfecting device, a temperature of 120 °C should not be exceeded.

Maintenance:

After each surgery, individual parts of the instrumentation must be checked. It is also necessary to check whether cutting instruments are blunting. In the event of damage, the respective parts must be supplied to the manufacturer for repair. After 10 to 20 successful surgeries, you are advised to contact the manufacturer for a maintenance check of the instrumentation.

Inspection and testing of function:

All parts of instrumentation must be checked before use.

Sterilization:

Unless otherwise stated, it is advisable to use steam sterilization in a steam sterilizer equipped with antibacterial filter at 121 °C, pressure 205 kPa for 20 minutes or at 134 °C, pressure 304 kPa for 10 minutes.

Other information:

The company MEDIN, a.s. has validated the above procedures for cleaning, disinfecting, drying and sterilization. Within the validation these procedures were recognized as eligible in the preparation of medical devices for reuse. User is responsible for ensuring the necessary materials, equipment and staff training in accordance with these codes of practice, to achieve the desired result. The user can also employ other methods, but the effectiveness of these alternative procedures must be verified by professional laboratory.

Storage:

The instrumentation must be stored in a dry place without direct exposure to chemicals.

Disposal

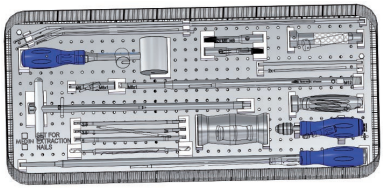
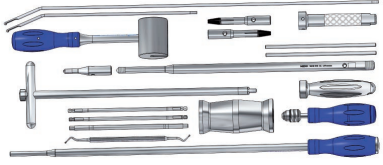
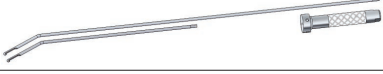
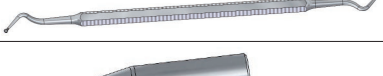
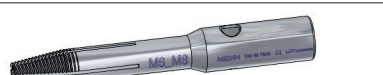
Instrumentation is considered a hazardous waste. User is responsible for implementing measures of safe handling and disposal of the product.

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