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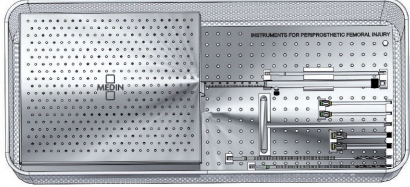
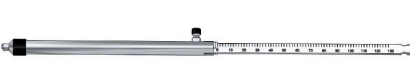
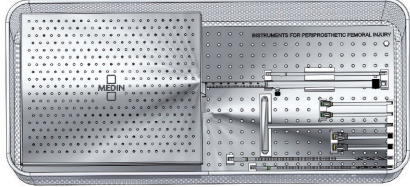
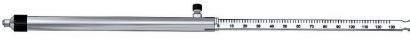
INSTRUMENTARIUM FOR PERIPROSTHETIC FEMORAL FRACTURES

Description of the instrumentation
The instrumentarium is intended to ensure the implants for periprosthetic femoral fractures.

Instrumentation comprises:

Pcs	Name	Image
1	Set instrumentarium for periprosthetic femoral fractures 397139090350	
1	Stand for screws 397129695680	
2	Guide sleeve 3,9/2,9 397129694430	
2	Drill sleeve 6,8/3,9×80 397129694300	
1	Depth gauge for screws 4,5 a 6,5 397129798910	
	Screwdriver T 3,5 397129694050	
1	Screwdriver hexagonal 3,5 397129695251	
1	Drill 2,9×190 397129799981	
1	Drill 3,9×220 397129695141	

Compatibility
The instrumentarium is used to with implants for periprosthetic femoral fractures simultaneously

PL0207 – Implants for periprosthetic femoral fractures

Indications
The instrumentarium is intended to ensure the implants for periprosthetic femoral fractures.

Contraindications
The instrumentarium must not be used if one or several instruments are damaged

Instructions for use

Preparation of instruments
Before the operation, it is necessary to check the completeness of the entire set of instruments and functionality (reliability performance) of all the instruments in the set.
The instrument set is used to for making connection between clamp and femoral proximal plate, femoral distal plate, or straight plate.
The hole for ø5/3.5 mm screw to clamp 1, 2 and 3 is pre-drilled by green colored drill ø2.9×190 mm.
The holes for ø4.5 mm cortical screws in clamp 3 are pre-drilled by drill ø3.9×220 mm

WARNING

Reprocessing limitation

Repeated processing has minimal effect on these instruments. End of life is normally determined by their wear and damage during use.

INSTRUCTIONS

Place of use:

Professional workplaces of health care providers, namely surgical, trauma and orthopedic departments.

Storage and transport:

Prevent deformation and scratches to surgical instruments; avoid their contact with other metallic materials and chemicals.

Preparation for cleaning

When cleaning and disinfecting, it is necessary to avoid agents containing chloride ions, which can cause corrosion of steel parts.

- 1. Mechanically clean with water using a brush.
- 2. Rinse instruments with pressurized water.
- 3. Leave the instruments in a disinfectant solution for 20–30 minutes. Recommended disinfectants are Sekusept Pulver, NeodisherseptoMED. In case of using another solution you risk damaging the instruments.
- 4. Rinse repeatedly with pressure water jet until there is no unpolluted water.

Disinfection:

We recommend either the equipment for washing and thermal or thermochemical disinfection or hand washing of instruments after chemical disinfection with virucidal agent, while the two stages can be combined using disinfectants with combined washing effect. The disinfectant may only be used in accordance with instructions on the label of the product. In the preparation of disinfectants and cleaning solutions, the procedures specified by the manufacturer must be followed. Upon performing disinfection, the instrument is to be rinsed with clean, demineralised water for removing residue remaining on the instrument.
Recommended products: Sekusept pulver, NeodisherseptoMED

Drying:

If drying is part of cycle for washing and disinfecting device, a temperature of 120 °C should not be exceeded.

Maintenance:

After each surgery, individual parts of the instrumentation must be checked. It is also necessary to check whether cutting instruments are blunting. In the event of damage, the respective parts must be supplied to the manufacturer for repair. After 10 to 20 successful surgeries, you are advised to contact the manufacturer for a maintenance check of the instrumentation.

Inspection and testing of function:

All parts of instrumentation must be checked before use.

Sterilization:

Unless otherwise stated, it is advisable to use steam sterilization in a steam sterilizer equipped with antibacterial filter at 121 °C, pressure 205 kPa for 20 minutes or at 134 °C, pressure 304 kPa for 10 minutes.

Other information:

The company MEDIN, a.s. has validated the above procedures for cleaning, disinfecting, drying and sterilization. Within the validation these procedures were recognized as eligible in the preparation of medical devices for reuse.
User is responsible for ensuring the necessary materials, equipment and staff training in accordance with these codes of practice, to achieve the desired result.
The user can also employ other methods, but the effectiveness of these alternative procedures must be verified by professional laboratory.

Storage:

The instrumentation must be stored in a dry place without direct exposure to chemicals.

Disposal

Instrumentation is considered a hazardous waste. User is responsible for implementing measures of safe handling and disposal of the product.

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